



# You're almost done!

Follow these simple steps to activate your MemorialCare Virtual Exam Kit. Please activate your device **within three days** of receiving it.

- 1** **Download** the TytoCare app.  
**Note:** Don't create a TytoCare account. It will be created automatically via your MemorialCare app account.
- 2** **Download the MemorialCare app** to your mobile device.
- 3** **Tap 'TytoCare Pairing & Resources'** in the MemorialCare app menu under Resources. This one-time step will open the TytoCare app on your mobile device.
- 4** **Follow the on-screen instructions** to complete setup. Setting up your device in advance allows you quick access to care when you need it.



For more information about your Virtual Exam Kit, scan the QR code or visit [memorialcare.org/virtual-exam-kit](https://memorialcare.org/virtual-exam-kit).

Need help? Call our 24/7 support line at 714-430-8237.




# Conduct a TytoCare Visit

Once you've completed the required activation (see Quick Start Guide on the reverse), you can carry out a video visit.

## 1 Complete TytoCare exams:

- Log in to the **TytoCare app**.
- On the home screen, tap '**Start**' and choose a patient.
- Follow the in-app steps to complete the exams based on your symptoms.

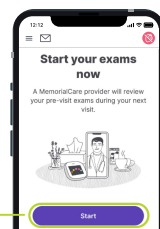

## 2 See a provider by scheduling a visit:

- Log in to your MemorialCare app. Select the **Schedule** button from the main footer and select "**Schedule a service.**"
- Select the visit type.
  - **To schedule a 24/7 Virtual Urgent Care Visit:** select "**TytoCare 24/7 Virtual Urgent Care**" from the Menu.
- Choose a time for your visit with a provider and schedule the appointment.
- Complete the Pre-Visit Check In by tapping the eCheck-In icon.



703-00226 A01

Need help? Call our 24/7 support line at 714-430-8237.